


Feline Interpretation Guide

Positive for Agglutination

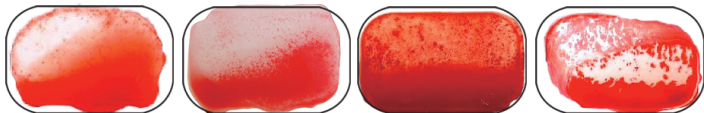

Example 1

Example 2

Example 3

Example 4

If the assay was run correctly, visible agglutination (see various positive examples above) should have occurred in at least one of the wells marked "Patient Test." The "Auto-Agglutination Pre-Screen" well should not show agglutination. (Auto-agglutination is a rare occurrence.)

If the patient sample shows agglutination in the well marked "Type A," the cat tested has blood group A. In this case, the well marked "Type B" will be negative for agglutination. (See negative example below.)

If the patient sample shows agglutination in the well marked "Type B," the cat tested has blood group B. In this case, the well marked "Type A" will be negative for agglutination.

If the patient sample shows agglutination in both patient wells, the cat tested has blood group AB.

Negative for Agglutination



dmslaboratories, inc.

2 Darts Mill Road, Flemington, NJ 08822

Tel.: (908) 782-3353 or (800) 567-4367 in U.S. and Canada; Fax: (908) 782-0832
RapidVet is a registered trademark of **dms**laboratories, inc.