

POLYGLYCAN®-SA

**(Hyaluronic Acid + Chondroitin sulfates C4 & C6
+ N-acetyl-D-glucosamine Sterile Solution)**

**STERILE POST-SURGICAL LAVAGE, SYNOVIAL FLUID REPLACEMENT,
VISCOSUPPLEMENT**

**Veterinary Use Only
FOR USE IN HORSES**

Patent Nos.: 6,979,679 / 7,485,629 / 8,455,458

INDICATIONS

Polyglycan®-SA is indicated for the replacement of lost or damaged synovial fluid. Polyglycan®-SA contains naturally occurring components of the synovia that play a central part in maintaining the homeostatic environment of the joint.

BENEFITS

- **Helps replenish** lost synovial fluid with naturally occurring elements
- **Assists** in the normalization of synovial fluid viscosity
- **Post surgical lavage** of joint cavity and surfaces
- **Patented formulation**



CAUTION

This device is restricted to use by or on the order of a licensed veterinarian.

PACKAGING

LIST NO.	UNIT PACKAGE	CASE SIZE
1POL009	6 x 2.5 mL vials	60

Bimeda, Inc.
One Tower Lane, Suite 2250
Oakbrook Terrace, IL 60181 USA
Toll Free Tel. 888-524-6332 • Toll Free Fax. 877-888-7035
Email: Sales@BimedaUS.com

See reverse side for additional information



BimedaUS.com

POLYGLYCAN®-SA

(Hyaluronic Acid + Chondroitin sulfates C4 & C6 + N-acetyl-D-glucosamine Sterile Solution)

STERILE POST-SURGICAL LAVAGE, SYNOVIAL FLUID REPLACEMENT, VISCOSUPPLEMENT

Veterinary Use Only
FOR USE IN HORSES

Patent Nos.: 6,979,679 / 7,485,629 / 8,455,458

CAUTION:

This device is restricted to use by or on the order of a licensed veterinarian.

DESCRIPTION:

Polyglycan®-SA is a patented formulation designed to replace lost or damaged synovial fluid. Polyglycan®-SA contains naturally occurring components of the synovia that play a central role in maintaining the homeostatic environment of the joint.

COMPOSITION:

Highly viscous aqueous solution of defined fractions of purified Hyaluronic acid, Chondroitin sulfates C4 & C6 in a 10% solution of N-acetyl-D-glucosamine.

EACH VIAL CONTAINS:

Hyaluronic acid sodium salt.....	12.5 mg
Sodium chondroitin sulfate.....	250 mg
N-acetyl-D-glucosamine.....	250 mg

STORAGE:

Refrigerate whenever possible. DO NOT FREEZE. Maintain product at or below 65°F (18°C) during shipment and storage.

PROPERTIES:

Glycosaminoglycans are important components of all extracellular tissue structures including cartilage and synovial fluid. The active components in Polyglycan®-SA exhibit viscoelastic and polyionic properties similar in nature to synovial fluid.

Introduce Polyglycan®-SA into the synovial space as a post-surgical lavage, synovial fluid replacement or viscosupplement to assist in the normalization of the joint.

DIRECTIONS:

Instill Polyglycan®-SA into the synovial space to replenish lost synovial fluid. Quantity to be determined by Veterinarian Surgeon. Discard any unused portion of vial contents.

WARNING:

Do not administer to animals that are to be slaughtered for use in food. Keep out of reach of children.



53 Industrial Circle ♦ Lancaster ♦ PA 17601 ♦ 1-800-233-0210 ♦ www.pennvet.com