

# I-TEAR<sup>®</sup> TEST



## TEAR VOLUME TEST

- ✓ Non-invasive
- ✓ Quick, 5-second test
- ✓ No anaesthetic required
- ✓ For professional-use only

A quick and easy test to measure the volume of tears produced. This quantitative measurement provides important information on the efficacy of the patient's lacrimal system as part of a complete KCS diagnosis protocol.


**1 box contains 50 test strips**  
*(One sterile strip tests 2 eyes)*



# I-TEAR<sup>®</sup> TEST


## HOW TO USE

- 1 Carefully open the transparent, sterile pouch at either end so as to not damage the device.
- 2 Hold the I-TEAR<sup>®</sup> Test strip in the center, as in the diagram below. Do not touch the tips of the device that contain the blue dye.
- 3 Apply the "R" marked tip of the I-TEAR<sup>®</sup> Test to the **Right** eye's lateral lower lid tear meniscus without touching the cornea and conjunctiva.

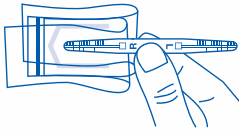

Contact with tear

- 4 Measure the length of the stained tear column, marking with a pen. A reading of 8 and over is normal, anything under 8 is deficient.

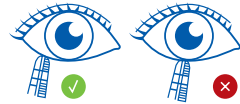

Avoid touching the cornea

- 5 Repeat the same procedure on the **left** eye using the "L" marked tip.



I-MED Animal Health innovates, manufactures and distributes products for the global veterinary medical and surgical eye care community.

T (514) 685-8118 | (800) 463-1008 | F (514) 685-8998  
info@imedpharma.com | [www.imedanimalhealth.com](http://www.imedanimalhealth.com)