

"Put an End to Anal Sac Disease"



ClinzGard NS[®]

Anal Sac Infusion Gel



(Clindamycin hydrochloride 1%)

Veterinary Medical Device - Non-Sterile Hydrogel

ClinzGard NS[®] is a one time application for dogs and cats with anal sac disease.

Infused into anal sacs, ClinzGard NS[®] gel thickens at body temperature and stays in place for 7-10 days while releasing Clindamycin HCl. Clinical studies also show that ClinzGard[®] decreases the size and bacterial load in abscess tissue and blood.

(See study summary at www.trillogicpharma.com)

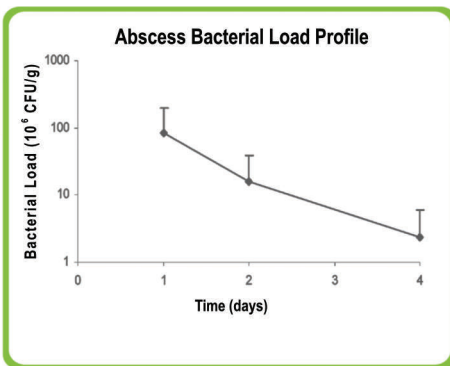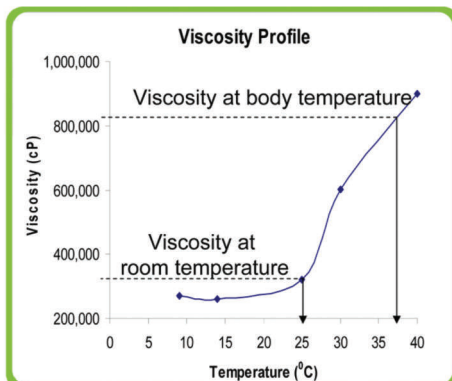

- Abscess size decreases over time
- Bacterial load decreases over time
 - 3 log reduction (99.9%) by day 4

Anal Sac Application

After expression of sacs, attach blunt nose cannula to syringe and fill sacs. Gel will thicken and stay in sacs for 7-10 days.



ClinzGard NS[®] can also be used in Tissue Abscesses, Bite and Puncture Wounds and Lacerations. Thoroughly clean and dry, debride if necessary, then apply ClinzGard NS[®] as the last step before suturing or bandaging.

- **Convenient**
Pre-filled 2.0 mL syringes require no mixing.
- **Easy to Use**
Specialized tip allows application into anal sacs or bite wounds.
- **Non-Refrigerated**
Two year shelf life at controlled room temperature (2° - 25° C)
- **Bio-adhesive and bio-eroding**
Through a process of reverse thermal gelation, ClinzGard NS[®] thickens in the sac or wound remaining in place for 7-10 days and acting as a barrier against bacteria. It is then bioresorbed by the body and eliminated through normal metabolic means. No removal necessary.
- **Sustained release**
of Clindamycin over 7-10 days.
- **Peace of Mind**
Send the patient home knowing that they have protection against infection.

Please read complete Instruction for Use (IFU) before using this product.



Supplied as 4 ea 2.0 mL Units with 8 blunt nose cannulas

TriLogic Pharma, LLC
5331 Perimeter Parkway Ct.
Montgomery, AL 36116

TRILOGICPHARMA
The Clear Solution Delivered

www.trillogicpharma.com — 866-562-8042 Toll Free





ClinzGard®

Barrier Gel (Clindamycin hydrochloride 1%)
Veterinary Medical Device


Sterile Surgical Gel for use in dogs and cats.

ClinzGard® is a sterilized viscous formulation intended for application in Surgical Incisions or Orthopedic Procedures.

To use in surgical incisions, apply ClinzGard® before wound closure. For orthopedic procedures, apply ClinzGard® to implant before insertion. In all applications, ClinzGard® is bioresorbed as the wound heals and eliminated through normal metabolic means.

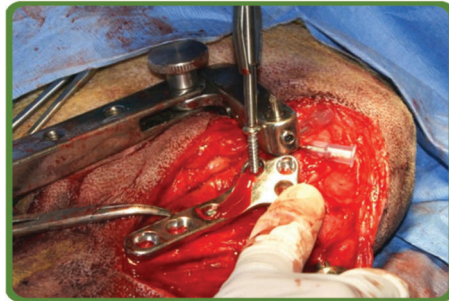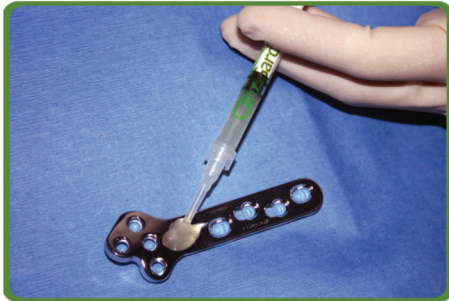

- **Convenient**
Can be used with other antibiotic therapy.
- **Easy to Use**
Sterile, single-dose syringes with conical tips allow application in hard to reach places.
- **Non-Refrigerated**
Two year shelf life at controlled room temperature (2° - 25° C)
- **Bio-adhesive and bio-eroding**
Through a process of reverse thermal gelation, ClinzGard® thickens in the crevice remaining in place for 7-10 days and acting as a barrier against bacteria. It is then bioresorbed as the wound heals and eliminated through normal metabolic means. No removal required.
- **Sustained release**
of Clindamycin over 7-10 days.
- **Peace of Mind**
Send the patient home knowing that they have protection against infection.

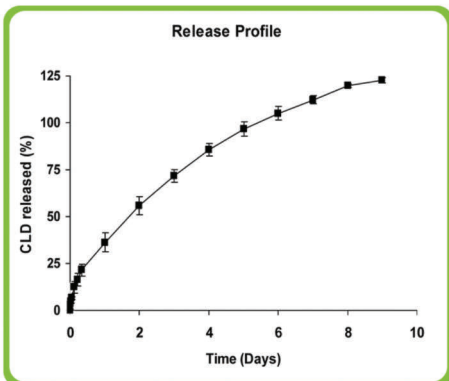

ClinzGard® stays in place for 7-10 days acting as a barrier against bacterial colonization.

Formulation	$t_{50\%}$ (Days)	$t_{100\%}$ (Days)	Viscosity (cp)
ClinzGard	2	7	825,000

Please read complete Instruction for Use (IFU) before using this product.



Supplied as 4 ea 1.0 mL Sterile Units with conical tips

TriLogic Pharma, LLC
5331 Perimeter Parkway Ct.
Montgomery, AL 36116

Available through Authorized Distributor Representatives
www.trilogicpharma.com — 866-562-8042 Toll Free

TRILOGICPHARMA
The Clear Solution Delivered