



CheckUp Kit Cat

Cat Urine Sample Collection Made Easy





How to Use Kit4Cat

For monitoring a specific chronic condition, visit www.checkupkit.com to purchase our diagnostic testing strips.



Pour the litter Pour all hydrophobic litter in a



Wait for your cat Allow your cat to urinate freely on the hydrophobic litter.



Collect sample Easily collect urine from the top of the litter with the pipette.



Fill the vial Fill the vial at least 3/4 way using the pipette.



Store the sample Close the vial with the cap and keep in refrigerator for up to 48h.



Further analysis Take the sample to your vet for further testing.

• For in vitro diagnostic use in animals only.

• Do not eat hydrophobic litter. • Avoid contact with eyes.

In the Box





3 x sample collection vial







Lab Validated & Vet Approved



Biodegradable



• Keep away from children. Manufactured by: Coastline Global, Inc. 827 Lincoln Ave, Suite B11, West Chester, PA 19380 888-999-3050 (tel) 888-999-3758 (fax) info@coastlineglobal.com



www.checkupkit.com