



CLEANING YOUR VETGUARDIAN DEVICE

Always follow the hygiene or infection-control rules of your organization as well as the instructions for use that come with the disinfectant.

With the following instructions, you can wipe the product's hard, nonporous surfaces gently with a disinfecting wipe or an approved cleaning solution and a sterile cloth.

- 1 Turn off the device. Unplug the device if it is connected to a charger.
 - 2 Use a disinfectant wipe that is already wet, or put the approved cleaning solution on a clean cloth if you are using a liquid. Wring out any excess liquid.
 - 3 Wipe all surfaces of the VetGuardian with the moistened cloth, following the manufacturer's instructions for use. Do not allow any abrasive material to touch these surfaces. Do not allow excess liquid to pool on the device.
 - 4 Keep a dry, sterile cloth ready to dry the device after cleaning.
 - 5 Allow to completely air dry before turning on the power and/or connecting it to a charger.
-

****WARNINGS/PRECAUTIONS****

- Do NOT use bleach.
- Do NOT let the cleaning solution pool on any part of the product to keep water from getting into ports or other openings. The button and connector areas require the most attention to prevent liquid from entering the device.
- Do NOT submerge the device in any liquid.
- Follow all cleaning solution manufacturers' instructions for use, including all warnings and precautions. The following is the list of approved cleaning solutions:
 - Isopropyl Alcohol less than or equal to 70%
 - Isopropyl Alcohol Wipes less than 70%
 - Clorox® Disinfecting Wipes Bleach-Free
 - Mild soap solutions