

## Ultra Scale™

Give your patients the care they **deserve.**



With companion animals, which are especially at risk for periodontal disease, it is important to take the measures necessary to keep your patients safe for the long term.

**Ultra Scale™** is a scaling concentrate that helps inhibit bacteria that can cause plaque and cavities. Ultra Scale™ can be used in all dental units and will not block or plug water lines of any dental unit.

### Ultra Scale™ Features

- Available in 8.5 oz / 250 ml container
- Chlorhexidine-free
- Contains special lubricant for scaling and cutting
- Removes biofilm from patient and equipment
- Mint smell and flavoring helps to mask mouth odor

### Directions

- Shake well before using
- Mix contents with 500 ml of distilled water
- Add solution to dental unit reservoir

*Note: Ultra Scale™ is light-sensitive. Please use a brown or opaque bottle. Solution must be used within 36 hours of mixing.*

### Storage Recommendations

- Store between 32° - 77° F (0° - 25° C)
- Keep out of direct sunlight