

## MPS - Medical Pet Shirt™

Length sizes are from nape of neck to base of the tail.

MPS DOG	Length	Dog breeds (reference only)
MPS-4XS	19 - 30 cm	Mini Chihuahua
MPS-3XS	22 - 35 cm	Chihuahua, Toy Poodle
MPS-2XS	33 - 42 cm	Lhasa, Yorkshire Terrier
MPS-XS	40 - 45 cm	Maltese, Boston Terrier
MPS-S	43 - 51 cm	Beagle, French Bulldog, Fox Terrier
MPS-S+	49 - 57 cm	Beagle, American Steffordshire, Chow-Chow
MPS-M	55 - 69 cm	Schnauzer, Samoyed
MPS-M+	61 - 72 cm	Labrador, Collie, Boxer
MPS-L	67 - 76 cm	Labrador, Rottweiler, German Sheperd
MPS-XL	74 - 82 cm	Bullmastiff, Rhodesian Ridgeback
MPS-2XL	80 - 92 cm	Leonberger, St. Bernard, Newfoundland

MPS CAT	Weight	MPS RABBIT	Weight	
MPS-4XS	25 - 32 cm	Young kitten	MPS-3XS	0,6 - 0,8 kg
MPS-3XS	29 - 37 cm	Kitten	MPS-2XS	0,8 - 1,5 kg
MPS-2XS	33 - 42 cm	Young cat	MPS-XS	1,5 - 2,0 kg
MPS-XS	40 - 45 cm	Cat	MPS-S	2,0 - 3,5 kg
MPS-S	43 - 51 cm	Cat	MPS-M	3,5 - 4,5 kg
			MPS-L	4,5 - 5,5 kg

## Important

The animal may need time to get used to the product. Distracting the animal with a toy can help. It is recommended to keep the animal supervised especially outdoors.



Professional wound management

Less stress, faster healing!



MPS - Medical Pet Shirt™2006



## MPS-TOP Shirt™

Chest circumference measurement right behind front legs.

MPS-TOP	Chest circumference
TOP-2XS	28 - 43 cm
TOP-XS	38 - 52 cm
TOP-S	48 - 68 cm
TOP-M	60 - 80 cm
TOP-L	72 - 92 cm
TOP-XL	78 - 100 cm

## MPS-HLS Hind Leg Sleeves™

Length sleeve measurement inside leg.

MPS-HLS	Length Sleeve
HLS-2XS	10 cm
HLS-XS	14 cm
HLS-S	20 cm
HLS-M	26 cm
HLS-L	32 cm
HLS-XL	37 cm

## Advantages

- Developed in collaboration with Veterinarians
- Adjustable with Velcro® and/or snaps
- Suitable for multiple purposes
- Easy to fit
- Machine washable at 30°C / 86° F
- Toxic free materials
- No child labour



## MPS-TOP Shirt "4 in 1"™2013



MPS-HLS Hind Leg Sleeves™2013

## MPS-TAZ Front Leg Sleeve™

Chest circumference measurement right behind front legs.

TAZ	TAZ2	Chest circumference	Length Sleeve
TAZ-3XS	TAZ2-3XS	27 - 33 cm	10 cm
TAZ-2XS	TAZ2-2XS	30 - 43 cm	12 cm
TAZ-XS	TAZ2-XS	40 - 52 cm	17 cm
TAZ-S	TAZ2-S	45 - 61 cm	23 cm
TAZ-M	TAZ2-M	54 - 68 cm	26 cm
TAZ-L	TAZ2-L	64 - 84 cm	35 cm
TAZ-XL	TAZ2-XL	80 - 104 cm	41 cm
TAZ-2XL	TAZ2-2XL	96 - 124 cm	48 cm

Products of Medical Pet Shirts International are model protected / patented by Medical Pet Shirts Europe B.V., The Netherlands

[www.medicalpetshirts.com](http://www.medicalpetshirts.com)  
[info@medicalpetshirts.com](mailto:info@medicalpetshirts.com)



v 3.0UK English

## MPS-TAZ Front Leg Sleeve™2012



## MPS - Medical Pet Shirt<sup>®</sup>™

The Medical Pet Shirt is a professional alternative to a collar and full body bandage for protecting wounds and bandages. The design is based on the animal's shape and anatomy with a snug fit as result.

Manufactured using a breathable stretchy Cotton / Lycra fabric, giving complete freedom of movement without compromising strength or durability. Thanks to the rear closure effortless to fit.



### Applications:

#### 1. Protection of wounds and bandages

- Gives warmth to the animal after surgery when waking up from anaesthetic.
- Helps to reduce stress, fear and uncertainty thanks to the snug and comfortable fit.

*"Less stress, faster healing!"*

- Inner fabric is light in colour to aid the inspection of any wound discharges.
- Double skin fabric across the belly is manufactured roomier to promote air circulation around the wound.
- Inner rear pouch for insertion of ice packs or pads can protect area in case of excessive licking.
- Blind buttonhole between shoulders present for passing an IV-line.

#### 2. In cases of skin diseases<sup>1</sup>

- Minimises injury to the skin due to scratching, licking or biting.
- Promotes faster healing, medication cannot be licked off of the skin.

#### 3. When in season or with incontinence

- When in season or with light to moderate incontinence, a hygiene pad can be inserted into the inside of the shirt.



*When walking the dog only the rear side of the shirt needs to be opened. The shirt can simply be rolled inwards at the belly side and fastened with the specially designed hold up system.*

*The majority of cats and rabbits are still able to use a litter tray unimpeded whilst wearing a shirt as the red (cats) and green (rabbits) shirts are designed with a larger opening under the tail.*

<sup>1</sup> After consulting Veterinarian

## MPS-TOP Shirt "4 in 1"<sup>®</sup>™

An ideal solution for wound protection providing comfort after surgery whilst allowing freedom of movement.

Manufactured using a soft double layered Cotton fabric around the body. The front side of the body is manufactured using a breathable stretchy Cotton / Lycra fabric with reinforced Cotton bands. Thanks to the adjustable closure around the chest fitting is effortless.



### Applications:

#### 1. Protection of wounds, bandages / in cases of skin diseases<sup>1</sup>

- Protects wounds and bandages specifically around the chest, shoulders and neck.
- Open hole at shoulders may be used to pass an IV-Line.
- Minimises injury to the skin due to scratching, licking or biting.
- Promotes faster healing in cases of skin conditions, medication cannot be licked off of the skin.

#### 2. Anxiety management

- Offers a gentle pressure around the chest thanks to the adjustable closure with elasticated lower band. This pressure has a calming effect for most animals especially useful for procedures such as knee surgeries when combined with the MPS-HLS Hind Leg Sleeves.
- Helps to reduce issues with barking, travel, fireworks or separation without the use of sedatives.

#### 3. Device holder

- Closable pocket on back (size 12(l) x 9(w) x 3(h) cm) to hold devices such as portable IV, Holter-ECG or suction pump.

#### 4. Fixation of MPS-HLS Hind Leg Sleeves (purchased separately)

- Keeps hind leg sleeves firm in place with the integrated loops on the rear side.



<sup>1</sup> After consulting Veterinarian

## MPS-HLS Hind Leg Sleeves<sup>®</sup>™

Wound and bandage protection and in cases of skin diseases whilst allowing freedom of movement.

Supplied as two sleeves left and right secured together with adjustable Velcro<sup>®</sup> and elastic straps to aid fitting. To ensure placement both hind leg sleeves need to be used, simply connect the integrated and adjustable elastic straps to the rear side loops of the MPS-TOP shirt, purchased separately.

The soft double layered fabric at the hips, back and front side of sleeve is manufactured using Cotton. The rear side of sleeve is manufactured using a breathable stretchy Cotton / Lycra fabric.

### Applications:

#### 1. Protection of wounds and bandages at the hind leg

#### 2. In cases of skin diseases<sup>1</sup>

- Minimises injury to the skin due to scratching, licking or biting.
- Promotes faster healing, medication cannot be licked off of the skin.



To ensure secure placement both hind leg sleeves need to be used!

## MPS-TAZ Front Leg Sleeve<sup>®</sup>™

Wound and bandage protection and in cases of skin conditions whilst allowing freedom of movement.

Soft double skin fabric around the body made from Cotton. The sleeve is manufactured with Cotton / Lycra stretch fabric. MPS-TAZ2 two legs Sleeve is provided with soft elbow pads. Thanks to the adjustable closure around the chest fitting is effortless.

### Applications:

#### 1. Protection of wounds and bandages at the front leg

#### 2. In cases of skin diseases<sup>1</sup>

- Minimises injury to the skin due to scratching, licking or biting.
- Promotes faster healing, medication cannot be licked of the skin.
- Protects and brings comfort with elbow diseases (MPS-TAZ2)



MPS-TAZ One leg Front Leg Sleeve, left / right exchangeable

MPS-TAZ2 Two legs Front Leg Sleeve

<sup>1</sup> After consulting Veterinarian