



ANAL SAC EXCISION KIT - GEN II



SAFE AND NONTOXIC ALTERNATIVE TREATMENT FOR REOCCURRING ANAL SAC CONDITIONS

Anal sac conditions such as impaction or abscess formation are a common, often reoccurring problem in small animal medicine. The surgical excision of both anal sacs can be elected as a permanent solution to these reoccurring issues. This excision can be a surgical challenge because of the collapsed nature of empty anal sacs, the risk of anal sphincter compromise, and the lack of discernable anatomical landmarks in a vascular surgical field.

The Gen II product uses a safe and nontoxic polymer gel that is infused as soft gel into the empty anal sac. The gel solidifies after 2-4 minutes in the anal sac. This provides an easily discernible mass that can be dissected out fairly readily. This reduces surgery time and the risk of leaving anal sac remnants.

FEATURES

- ▶ 18g plastic blunt angled needle for easy insertion into anal sac opening/duct
- ▶ Dual chambered cartridge of polymer gel with mixing tip
 - ▶ Gel solidifies 2 to 4 minutes upon application
- ▶ 2.4cc capacity is sufficient for larger patients
- ▶ Superb ergonomic design for small hands
- ▶ Simple turn and click of syringe hub opens gel pathway
- ▶ Includes directions for use

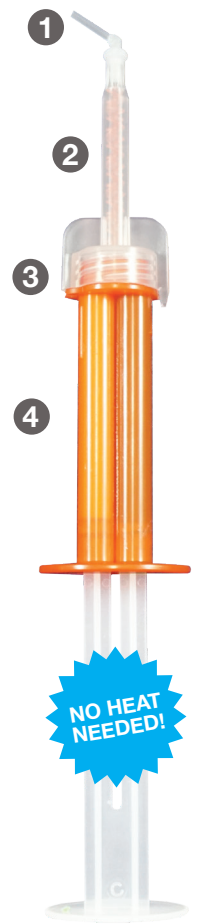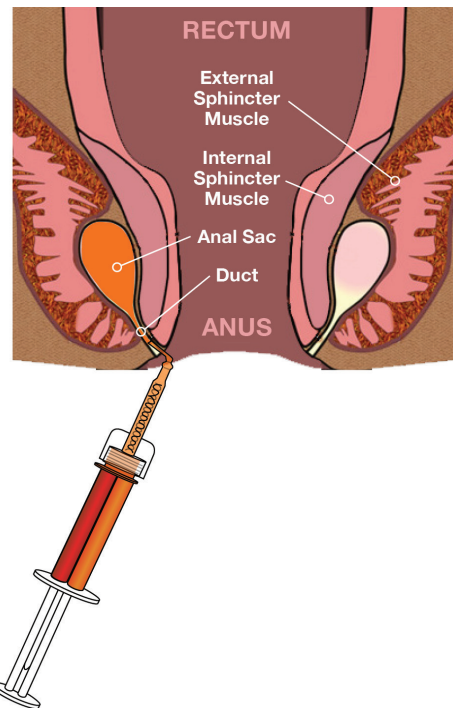

- 1 18g Rotating Blunt Angled Tip
- 2 Mixing Tip
- 3 Locking Head Twist for Activation
- 4 Dual Chamber Cartridge

7" Total Length

SKU	Description	Qty
J1490	Gen II Anal Sac Excision Kit - Single Complete Unit	5/Pk



Phone - 800.233.0210

www.pennvet.com

Jorgensen Laboratories, LLC
 800.525.5614 | fax 970.663.5042
 www.jorvet.com | info@jorvet.com