**IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL
MODEL SKU: VET-800NA**

Veterinary Forced Air Animal Warmer

**This device is designed for animal and pet warming use only.
It is not for use in human households or human medical applications.**

SAVE THESE INSTRUCTIONS.

READ CAREFULLY BEFORE ATTEMPTING TO ASSEMBLE,
INSTALL, OPERATE, OR MAINTAIN THE PRODUCT DESCRIBED.
PROTECT YOURSELF AND OTHERS BY OBSERVING ALL SAFETY
INFORMATION. FAILURE TO COMPLY WITH INSTRUCTIONS COULD
RESULT IN PERSONAL INJURY AND/OR PROPERTY DAMAGE!
RETAIN INSTRUCTIONS FOR FUTURE REFERENCE.

IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL

MODEL SKU: VET-800NA

IMPORTANT INSTRUCTIONS

DANGER: TO REDUCE RISKS OF BURNS, ELECTRIC SHOCK, FIRE, AND PERSONAL INJURY, THIS PRODUCT MUST BE USED IN ACCORDANCE WITH THE FOLLOWING INSTRUCTIONS.

WARNING

- **ANIMAL USE ONLY — DO NOT USE THIS WARMING SYSTEM FOR ANY HUMAN APPLICATIONS**
- **DO NOT USE FOR ANY HUMAN MEDICAL, OXYGEN, OR WET ENVIRONMENTS**
- This appliance is not intended for use by persons (including children) with reduced physical sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the appliance.
- **CAUTION** — Some parts of this product can become very hot and cause burns. Particular attention has to be given where children and vulnerable people are present.
- Heater must not be located immediately below a socket outlet.
- Do not use this heater in the immediate surroundings of a bath, a shower or a swimming pool.
- If the supply cord is damaged, it must be replaced by the manufacturer, its service agent, or similarly qualified persons in order to avoid a hazard.
- **DO NOT cover** 

1. Read all instructions before using this device. Use this device only as described in the manual. Any other use not recommended by the manufacturer may cause fire, electrical shock, or injury to persons.
2. Do not operate device unattended. Turn off or unplug when not in use.
3. Make certain that the power source conforms to the electrical requirements of this device.
4. **DO NOT OPERATE MORE THAN ONE BEDJET ON SAME AC-POWER OUTLET WITHOUT SETTING LOW POWER SWITCH TO ON — SWITCH LOCATED ON REAR OF DEVICE.**
5. This appliance has a polarized plug (one blade is wider than the other). To reduce the risk of electric shock, this plug is intended to fit in a polarized outlet only one way. If the plug does

IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL

MODEL SKU: VET-800NA

not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. Do not attempt to defeat this safety feature.

6. DO NOT plug device into any other cord connected device, such as a power strip, surge protector, multiple outlet adapters, or outlet-type air fresheners. The use of such devices may create a fire hazard. Plug unit directly into a wall outlet only.
7. Always plug this device directly into a wall outlet/receptacle. Never use with an extension cord or relocatable power tap (outlet/power strip).
8. Do not operate this device with a damaged cord or plug, or after the device malfunctions or has been dropped or damaged in any manner. Return device to authorized service facility for examination, electrical, or mechanical adjustment or repair.
9. To disconnect device, make sure device is not currently operating in heat or cool mode, press the Main Power Button, then remove plug from outlet.
10. Always unplug device when not in regular use. Unplug power cord before servicing or moving the device. **WARNING: DO NOT DEPEND ON THE ON/OFF SWITCH AS THE SOLE MEANS OF DISCONNECTING POWER WHEN SERVICING OR MOVING THE DEVICE. ALWAYS UNPLUG THE POWER CORD.**
11. Do not insert or allow fingers or foreign objects to enter any ventilation or exhaust opening as it may cause an electric shock, fire, or damage the device. Do not block or tamper with the device in any manner while it is in operation.
12. To prevent a possible fire, do not block air intakes or exhaust in any manner. Do not place unit on soft surfaces, like a bed, where openings may become blocked.
13. Always place the device on a stable, flat, level surface when operating, to avoid the chance of the device overturning. Locate the power cord so the device or other objects are not resting on it. Do not run cord under carpeting. Do not route cord under furniture or appliances. Arrange cord away from traffic area and where it will not be tripped over.
14. This device has hot and arcing or sparking parts inside. Do not use the BedJet in areas where gasoline, paint, or flammable liquids or flammable gases are used or stored. It must NOT be used in potentially dangerous locations such as flammable, explosive, chemical-laden, or wet atmospheres.
15. This device is not intended for use in wet or damp locations. This device is **NOT INTENDED** for use in bathrooms, laundry areas, and similar indoor locations. Never locate the device where it may fall into a bathtub or other water container. Do not immerse BedJet in water.
16. This device is not intended for use as a room air heater.
17. Do not use device outdoors.
18. Keep the cord away from heated surfaces.
19. Do not operate where aerosol (spray) products are being used or in oxygen rich environments.

IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL

MODEL SKU: VET-800NA

20. Check your BedJet cord and plug connections:

- a) Faulty wall outlet connections or loose plugs can cause the outlet or plug to overheat. Be sure the plug fits tight in the outlet.
- b) Heaters draw more current than small appliances; overheating of the outlet may occur even if it has not occurred with the use of other appliances.
- c) During use check frequently to determine if your plug outlet or faceplate is HOT! If so, discontinue use of the heater and have a qualified electrician check and/or replace the faulty outlets.

WARNING: REDUCE THE RISK OF FIRE OR ELECTRIC SHOCK – DO NOT USE THIS DEVICE WITH ANY SOLID-STATE SPEED CONTROL DEVICES.

SAVE THESE SAFETY INSTRUCTIONS.

MANUFACTURER INFORMATION:

BEDJET LLC
17 Connell Highway
Newport, RI 02840 USA
Phone: +1-401-404-5250
Email: info@bedjet.com

IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL

MODEL SKU: VET-800NA

Pre-Installation Checklist

Before attempting installation of the BedJet VET, please verify the following parts list from your boxed BedJet package:

Air hose with fabric liner, stretchable, compressible, and bendable

BedJet VET with pre-installed HEPA filter

Wireless radio remote control with 2 AA alkaline batteries (included)

Proper Assembly Required:

All parts included are required to be fully assembled and installed onto the BedJet for proper operation. Do not operate the BedJet without the included Air Hose. Do not attempt to use the BedJet with air delivery accessories or air delivery components not approved or supplied by BEDJET LLC.

IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL

MODEL SKU: VET-800NA

System Operation

ON/OFF Switch

Low Power Switch for setups with two BedJets operating on same AC circuit. **SEE SAFETY DETAILS BELOW.**

Status Display Ring

Expansion Port for factory testing, not a functional USB port

Low Power Switch:

Setups with two BedJet VET units plugged into the same AC circuit should set Low Power switch ON to avoid tripping/overloading the circuit breaker. Installations where only one BedJet VET is used in a room should leave Low Power Switch OFF to utilize the full power of the system.

⚠ WARNING: Failure to comply with the correct Low Power ON switch setting with more than one BedJet installed on the same room AC branch circuit may result in overloaded electrical circuit, circuit breaker trips, or electrical hazard. Never use more than one BedJet on the same AC circuit without Low Power switch ON.


IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL

MODEL SKU: VET-800NA

BedJet VET Wireless Remote Control:

All BedJet VET operating modes are activated via the wireless radio remote control, there are no exterior buttons located on the BedJet VET base unit for manual operation. With charged batteries installed, the remote control keys will light up automatically when picked up, indicating remote is ready for use. The BedJet VET will beep to acknowledge any command received from the remote control. The wireless remote control is pre-paired to your BedJet VET unit at the factory. However, if your BedJet VET is verified powered ON (power button LED is ON) and wireless remote buttons light up and are pressed but the BedJet does not activate, then re-pair the remote control to base unit.

Pairing a New Remote: Press and hold the “Low Power” button on the back display of the BedJet VET until it blinks, indicating the unit is in pairing mode. Press and hold the OFF key on your remote control to complete the pairing. Successful pairing will be acknowledged by three beeps.

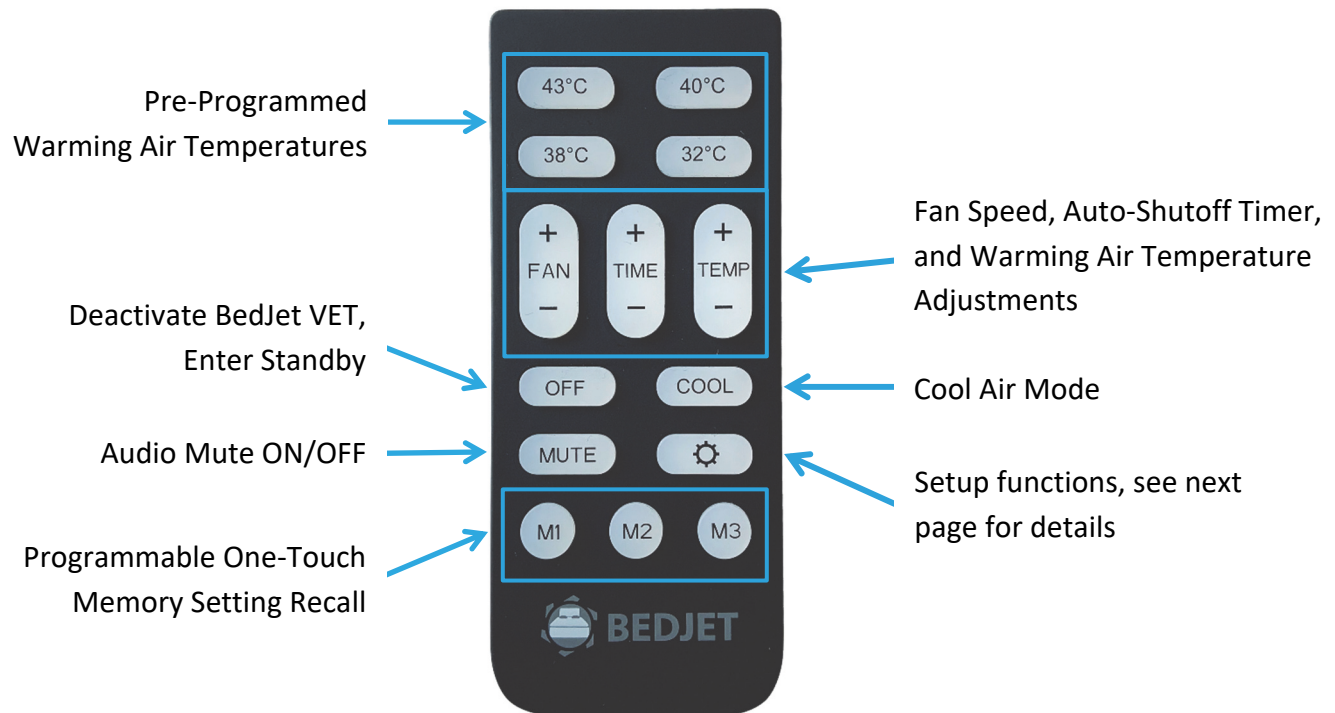IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL

MODEL SKU: VET-800NA

Warming Temperature Adjustments: Activate warming air mode by pressing any temperature preset button. Once active, warming air temperature can be further adjusted by 1°C increments from 43°C to 32°C using the +/- buttons. Audio announcement indicates current temperature setpoint.

Auto-shutoff Timer Adjustment: Activation of any air mode defaults the auto-shutoff timer to 4 hours. Once active, auto-shutoff time can be further adjusted by 30-minute increments using the +/- buttons. 4 hours auto-shutoff is the maximum allowable time setting. Audio announcement indicates current auto-shutoff timer setting.

Fan Speed Adjustments: Activation of any mode defaults to 100% maximum fan speed. For smaller animals with smaller warming bags, the fan speed may be reduced to lower the noise of the unit. The unit has 20 fan speed settings and will stop beeping when upper or lower fan speed limits are reached using the +/- buttons.

Programmable One-Touch Memory Recall: Frequently used settings can be recalled using the one-touch memory buttons. Simply turn the unit on and set to preferred temperature, fan speed and auto-shutoff settings and then press and hold any memory key for 3 seconds to save. When the unit is in standby, press and quickly release the same memory key to activate the unit and recall the saved settings.

HEPA Filter Change Reset: The unit contains a permanent service clock of operating hours. When 400-hour intervals are reached, a “CHANGE FILTER” audio alert will be heard. Reset the “CHANGE FILTER” audio alert after the HEPA filter has been changed (See Maintenance section) by simultaneously pressing the setup button (gear icon) and OFF button the remote control.

Status Display Ring Indicator

Solid Red = Operating at set air temperature

Spinning Red = Adjusting to reach set air temperature

Solid Blue = Cool mode

IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL

MODEL SKU: VET-800NA

Maintenance

The only maintenance required for the BedJet VET is periodic replacement of the HEPA filter and periodic cleaning of the washable HEPA prefilter.


HEPA PREFILTER CLEANING: The washable black foam prefilter is installed on top of the HEPA filter and helps extend the life of your HEPA filter. Clean the HEPA lifetime washable pre-filter every 2 months or when visually dirty. Simply remove the black foam prefilter from HEPA filter (it is attached to front of HEPA filter with Velcro) and clean under running water. Reinstall prefilter when dry and gently push the grill until it snaps back in place.

HEPA FILTER REPLACEMENT: The HEPA filter is recommended to be replaced every 400 hours or every 6 months of regular use. The BedJet VET contains a HEPA filter service clock. At 400 hours the BedJet VET will give an audible alert reminder to replace the HEPA filter.


HEPA FILTER SERVICE ALARM RESET: Hold down the gear icon button on the remote control while simultaneously pressing the OFF button. The HEPA filter service alarm should now be reset for an additional 400 hours. The service alarm can also be reset from the Bluetooth App.

Push both filter buttons (on either side of grille) in the direction indicated to release front air grille for easy access to filters.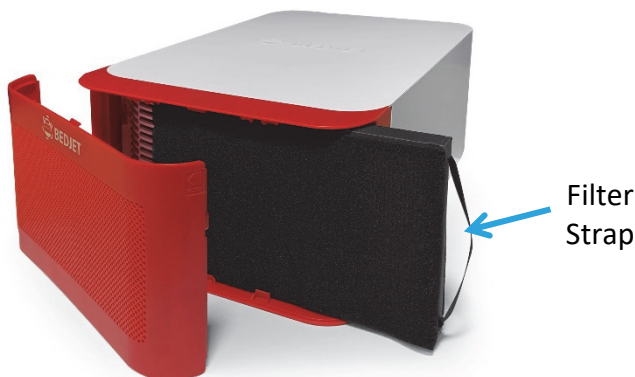

Once grille is opened, pull strap to slide filter assembly out.

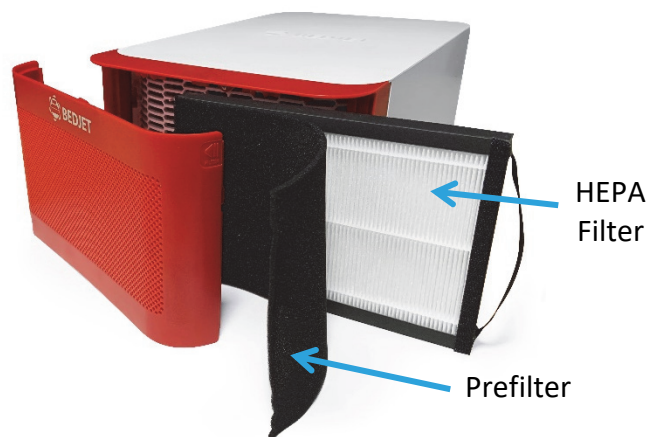
Peel black prefilter away from HEPA filter to separate for cleaning and/or replacement.

IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL

MODEL SKU: VET-800NA

Bluetooth Remote Control App

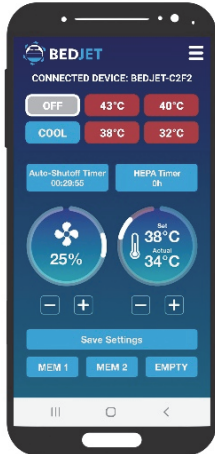Pending release (target: Q1 of 2022), the BedJet VET App can control all functions of the unit from a smartphone or tablet. Additional features in the app not available with the wireless remote include:

- HEPA filter life showing hours left to changeout
- Live output air temperature readings
- Live auto-shutoff timer readout and fan speed readout
- Firmware update check via WiFi
- Please note, the legacy BedJet VET V2 App currently available on iOS is not compatible with the new model BedJet VET. This legacy app can be identified by the older red/orange BedJet logo.

Troubleshooting

Use this guide to address problems with your BedJet VET product. If problems cannot be resolved, please contact Customer Service at info@bedjet.com or +1 (401) 404-5250. Do not return any product requiring service without first obtaining a Return Authorization Number (RMA) and shipping instructions.

BedJet VET Audible Notifications – What does the BEEP mean?

Issue	Indication	Recommendation
1 Beep	<ul style="list-style-type: none"> • BedJet switched on or off • Any command received from the remote control or App 	<ul style="list-style-type: none"> • This is normal operation
3 Beeps	<ul style="list-style-type: none"> • Remote control learning/pairing function has been successfully accomplished 	<ul style="list-style-type: none"> • This indicates successful learning/pairing of a remote control to the BedJet • Remote control has not been paired if 3 beeps are not made
5 Beeps	<ul style="list-style-type: none"> • A system monitoring error has occurred • The BedJet will be non-functional until resolved 	<ul style="list-style-type: none"> • See diagnostic table below, can only be reset from error state by unplugging for 30 seconds

IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL

MODEL SKU: VET-800NA

Troubleshooting Cont.

Diagnose & Fix

Issue	Possible Causes	Action
Remote control will not operate BedJet	<ul style="list-style-type: none"> • Dead battery in remote • Remote control may need to be re-paired to your BedJet 	<ul style="list-style-type: none"> • Make sure remote control LED lights up when picked up, if not then replace battery • Re-pair remote control to BedJet with instructions on page 7
Remote control commands are not acknowledged with an audible beep anymore	<ul style="list-style-type: none"> • Audio alert mute may have been accidentally turned on 	<ul style="list-style-type: none"> • Press Mute button on remote control to unmute
BedJet keeps shutting off without a shutoff command being given or beep; BedJet power button LED is ON and ERROR LED is NOT flashing and BedJet can be restarted normally	<ul style="list-style-type: none"> • This is normal. Once turned on, every mode has an automatic shutoff timer. Please see page 8 for information about auto-shutoff time limits. 	<ul style="list-style-type: none"> • Shutoff Timer can be advanced within certain limits using TIME +/- button
BedJet ERROR display LED is flashing and unit will no longer operate, BedJet beeps 5 times	<ul style="list-style-type: none"> • The BedJet microprocessor has discovered a monitoring error within the system. This does not always mean your BedJet is malfunctioning and may occur if you rapidly change modes or airflow while at the same time obstructing inlets or outlets. 	<ul style="list-style-type: none"> • Check to make sure air filter is clean and there are no air inlet or outlet obstructions. • Unplug BedJet for 30 seconds, run in cool mode for 2 minutes and then retry warming mode. If this does not solve the problem, or if the problem occurs again, contact BedJet support and record the number of blinking flashes from the Error display LED.
BedJet delivered warm air but then while still in warming mode is blowing cold air	<ul style="list-style-type: none"> • In addition to real-time microprocessor control of the heat, the BedJet contains multiple mechanical-thermal overheating switches for your protection. They are extremely sensitive and may have disabled the heater unit. They are self-resetting but remain open until power is removed and the unit is allowed to cool off. They may have been triggered by inlet or outlet blockages. 	<ul style="list-style-type: none"> • Check to make sure air filter is clean and there are no air inlet or outlet obstructions. • Unplug BedJet for 30 seconds, run in cool mode for 2 minutes and then retry warming mode. If this does not solve the problem, or if the problem occurs again, contact BedJet support.

IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL

MODEL SKU: VET-800NA

Specifications

- Voltage: **120V ONLY** (Not a dual voltage device)
- Power Rating: 1500 watts
- Heating Element Rating: 1440 watts
- Amp Rating: 12.8 Amps
- Rated operating temperature range 0°C to 40°C.
- HEPA Air Filter Type: .3μ replaceable
- Recommended filter change: 6 months or 400 hours
- Thermal Sensors: 2, dual redundant with microprocessor error checking
- Temperature Regulation Accuracy: ± 1.5 °C (as new)
- System Time to 38°C: ~16 seconds
- Dimensions: 17" x 10.3" x 5.8" high (not including hose outlet)
- Weight: 9 lbs. not including hose
- Hose length: 45" (hose extensions available)
- Remote Control Battery Type: AA x 2, alkaline type

Thermal Safety

The BedJet VET includes the most robust set of redundant safety features of any forced air animal warmer currently on the market to prevent unintentional overheating. These features include:

- Easily set Auto-shutoff times from 30 minutes to 4 hours
- Dual temperature sensors with microprocessor monitoring for error check — sensors are averaged for temperature readings and must agree with each other or the system will log a fault and shut down
- Dual microprocessors with error monitoring and watchdog functions
- 3-stage microprocessor monitored overheat protection that will shut down heat under the following out of specification over-temperature situations:
 - Air exhaust measured at > 46°C for 120 seconds
 - Air exhaust measured at > 53°C for 12 seconds
 - Air exhaust measured at > 55°C for 1 second
- Mechanical thermostatic overheat switch located in air exhaust that will shut off power to the heater as redundant safety feature to microprocessor monitoring failure

IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL

MODEL SKU: VET-800NA

Certifications

The BedJet VET has been designed to operate as a safe forced air heating device for animal warming purposes. This product has been rigorously tested by an accredited 3rd party testing lab to be qualified as a safe electric appliance. The BedJet VET complies with the following safety and product standards:

- Underwriters Laboratories 499
- Canada CSA Standard 22.2 No. 64

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device contains license-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's license-exempt RSS(s). Operation is subject to the following two conditions:

1. This device may not cause interference.
2. This device must accept any interference, including interference that may cause undesired operation of the device.

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IMPORTANT INSTRUCTIONS: USER OPERATING MANUAL

MODEL SKU: VET-800NA

Warranty

BEDJET LLC gives to the original purchaser (“the purchaser”) of a new BedJet VET (“the product”) this limited express warranty which covers any defects in materials or workmanship in the product under normal use. Except for this limited express warranty and any implied warranties under applicable law, BEDJET LLC makes no express or implied representations or warranties, either in this document or in any of its instruction manuals, product packaging, or other materials accompanying or used in conjunction with sale or marketing of the product.

DURATION OF WARRANTY

This limited express warranty, and any implied warranties (including implied warranties of merchantability and fitness for a particular purpose) on the product ***will be for two (2) years from the documented date of purchase***, as noted below. Coverage terminates if the purchaser sells, disposes of, or otherwise transfers the product.

During the two years following the documented date of purchase, BEDJET LLC will repair or replace any product or part(s) that are defective in either materials or workmanship to include parts, labor, and delivery back to the purchaser at the expense of BEDJET LLC. Delivery charges are covered under this warranty only for the continental United States.

EXCLUSIONS OF COVERAGE

This limited express warranty is conditioned upon proper use of the product by the purchaser. Any problem with the product that is caused by abuse, misuse, and operation contrary to or inconsistent with the written instructions or an act of God (such as a flood) is not covered. BEDJET LLC does not have any obligations if our inspection reveals that the product has been altered, modified, damaged, subjected to or involved in an accident, disassembled, repaired or attempted to be repaired by someone other than us, or used in a manner inconsistent with its intended purpose.

Indirect, special, incidental, or consequential damages (including any lost revenues, loss of use or lost profits) are not recoverable under the warranty, or with respect to the sale or use of the product. In no event will our entire liability to you exceed the purchase price actually paid by you for the product, or any defective portion thereof, whichever is less. This limitation does not apply to any claims for personal injury. Also, because some states do not allow, or restrict, limitations of incidental or consequential damages, some or all the foregoing may not apply to you.

What Are Your Rights Under State Law and Other Considerations?

This warranty, and any unresolved claims and disputes between you and BEDJET LLC shall be governed by and subject to the laws (exclusive of conflict-of-law provisions) of the State of Rhode Island, which shall also be the exclusive venue and jurisdiction for all such claims and disputes. All claims against BEDJET LLC must be brought within three (3) months from the time they arise. As some states do not allow, or restrict, limitations on how long an implied warranty lasts, some or all of the foregoing may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state. Other than as expressly set forth above, BEDJET LLC does not authorize any agent or representative to assume any obligation or liability here under. ***IF YOU DO NOT AGREE WITH THE ABOVE TERMS YOU MAY RETURN YOUR PRODUCT FOR A FULL REFUND WITHIN SEVEN DAYS AFTER PURCHASE.***

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