

Smile!

Assisi DentaLOOP™ Is Here

ASSISI DentaLOOP™

Assisi's powerful new product uses clinically-proven tPEMF™ technology to:

Reduce pain and inflammation

Improve affected tissues

Speed recovery time

Simple to operate
for easy at-home
treatments

Delivering the same proven effective, FDA-cleared tPEMF signal as the Assisi LOOP®, the new Assisi DentaLOOP™ device is specially designed for use on dental and oral surgery patients.

Benefits of Assisi DentaLOOP:

- Helps reduce oral and dental pain and inflammation
- Helps reduce pain and swelling after dental procedures, orofacial surgery, and trauma
- Reduces need for opioids and other pain medications
- Aids in preparation of oral tissues for surgery

Use the Assisi DentaLOOP to help:

- Speed recovery and reduce pain after dental extractions
- Reduce healing time and pain after orofacial surgery
- Reduce inflammation caused by stomatitis and other oral conditions
- Reduce swelling and improve tissue condition prior to oral surgery

Assisi DentaLOOP Quick Facts:

Operation: Auto turn-off

Hands Free: With DentaLOOP™ Bonnet

Number of Treatments: 60, 15-minute sessions

Add the optional Bonnet for hands-free operation (see on back)

ASSISI DentaLOOP™

Add the Convenience of Hands-Free Healing

The **Assisi DentaLOOP™ Bonnet** offers a comfortable and convenient hands-free treatment option, sold separately.

- Available in small, medium, and large sizes
- Multiple placement options for optimal facial positioning
- Made from soft fabric
- Worn similar to an Elizabethan collar



Pre-Surgery Protocol

Use DentaLOOP around the patient's injured, inflamed, or painful area of the mouth. Two 15 minute sessions per day helps reduce pain and inflammation, and improves affected tissue in preparation for a dental or orofacial procedure.

Post-Surgery or Post-Trauma Protocol


Apply the device around the patient's muzzle or adjacent to the injured or affected area two to four times a day for 15 minutes to help reduce edema and pain, and hasten healing.

tPEMF™ Therapy Explained

Assisi Animal Health's targeted Pulsed Electromagnetic Field therapy uses electromagnetic waves developed specifically to stimulate an animal's natural recovery process.

Assisi's peer-reviewed tPEMF™ signal enhances nitric oxide (NO) production, a key element in the healing process.

By increasing endothelial NO (eNO), the Assisi tPEMF signal activates anti-inflammatory cascades.

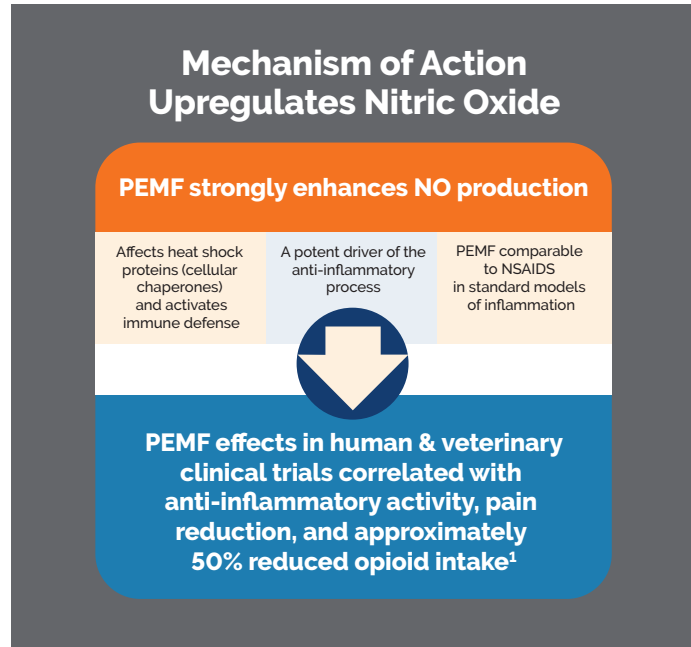

1. Alvarez LX, McCue J, Lam NK, Fox P, Gulce A. 2019. Effect of Targeted Pulsed Electromagnetic Field Therapy on Canine Postoperative Hemilaminectomy: A Double-Blind, Randomized, Placebo-Controlled Clinical Trial. *J Am Anim Hosp Assoc.*, 55(2): 83-91.

Reduce Pain and Speed Recovery Time in Your Dental Patients with Assisi DentaLOOP



Veterinary Supply, Inc.

Phone: 800.233.0210

www.pennvet.com



WHERE SCIENCE MEETS LOVE®

Learn more at:

assisianimalhealth.com | 866-830-7342

Assisi Animal Health is a member of

