


Why do veterinarians love Dentanomic instruments?



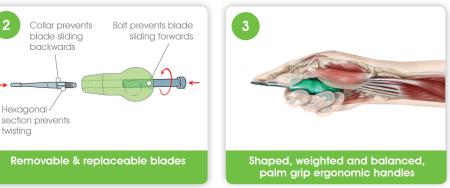

Connection is standardised any Dentanomic blade will fit any Dentanomic handle.



PALM GRIP PREVENTS THE INSTRUMENT SLIPPING AND STABBING YOUR PATIENT.

Available in 5 stunning colors, short & long version for glove sizes from 5 to 9!

Hardened blades stay sharp for longer than cheap instruments and can also be honed

Cheap dental instruments make it difficult to extract teeth.


They don't stay sharp, don't have replaceable blades and aren't thin enough to easily enter the periodontal space.

Our blade types:

SURGERY.

Elevation blades Stretch periodontal ligament. Use on thicker, stronger roots.


- Wedge the blade into the periodontal space.
- Gently rotate the blade and hold steady pressure for 10-30 seconds to stretch the ligament.

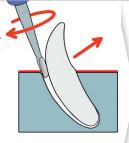

Luxation blades Cut periodontal ligament.

Cut periodontal ligament.
Use on thinner, weaker roots.

- Insert blade in the periodontal space.
- Rock blade sideways into periodontal space. Do NOT pry the blade out or away from the tooth.
- 3. Simultaneously contour the blade around the root.

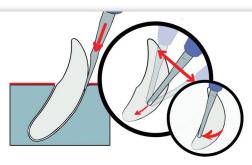

Using dental instruments as a prybar will damage both the instrument and the patient. If a surgeon uses correct surgical techniques, surgical performance and instrument life will be increased. More detail on correct surgical technique can be found at www.dentanomic.com.

Dentanomic™ is a trade mark of Bonovate Limited. Patent & registered design applied for.
©2019 Bonovate Limited, Unit 1b, Stanhope Gate, Stanhope Road, Camberley, Surrey, GU15 3DW. Registered number 7311740. Designed and manufactured in Britain.





"When we got a set of Dentanomic instruments, I could not believe the difference it made. Suddenly, extractions are faster and so much easier than before. Having great instruments has helped my confidence with dentals a lot; I no longer dread them!"

Bea, UK small animal veterinarian

Why Dentanomic?

O1 EASIER

O2 FASTER 03 SAFER 04

Bettei

It's much easier to cut ligament with our precision blades, than crush bone with cheap, blunt instruments.

No time for honing? New blades are not expensive and just slot into your existing handles. Blades should last for months even without maintenance (depending on surgical technique)

You can make your blades last longer by using our honing kit after each use.

Our customers report >30% faster extractions compared with using their normal instruments.

Shorter anaesthetics, better for geriatric patients & reduces risk of complications like hypothermia.

More ops/day or more chance of a lunch break.

Precise blades and ergonomic palm grip handle reduce risk of slipping & stabbing patient.

Reduce risk of hand strain – choose from small or large balanced handle options.

More precise surgical technique results in less tissue damage, less pain and better recovery.

~3/4 of pet dogs & cats have painful dental disease. Dental surgery is also a major driver of clinic profit that can't go online.

Easier, faster & safer dental extraction makes it more likely that veterinarians will recommend dental procedures.

Better instruments make it easier for you to provide the best care for your patients & enjoy and take pride in your dental surgery.

DENTANOMIC PRODUCT RANGE Blue Short Handle Green Glove size < 7.5 Pink Purple Long Handle Glove size > 7.5Gold 2mm 3mmElevation Blades 4mm 5mm 1.5mm 2mm 3mm Luxation Blades 4mm 5mm

Accessories	Blade Tray	
	Handle tray (Coming Soon)	
	Honing Kit	
Cat Starter Kit	Handle x1, E2, E3, L2	
Dog Starter Kit	Handle x1, E3, E4, L3	
Clinic Kit	Handle x3, All 9 blades, Blade Tray	
Specialist Kit	Handle x9, All 9 blades, Blade Tray	

WWW.DENTANOMIC.COM